FLOWline®

(DE)	Gebrauchsanweisung	3
(GB)	Instructions for use	7
(FR)	Mode d'emploi	11
(ES)	Instrucciones de uso.....	15
(IT)	Istruzioni per l'uso	19
(PT)	Instruções de uso	23
(NL)	Gebruiksaanwijzing	27
(SE)	Bruksanvisning	31
(DK)	Brugervejledning	35
(NO)	Bruksanvisning	39
(FI)	Käyttöohjeet	43
(GR)	Οδηγίες χρήσης	47
(TR)	Kullanma talimatı	52

KULZER
MITSUBI CHEMICALS GROUP

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Lightcuring hybrid composite of low viscosity, radio-opaque.

Shades:

A1, A2, A3, A3.5, B2, B3, OA2, OA4, OB2

Composition:

FLOW LINE is based on multifunctional methacrylate monomers (EBADMA/TEGDMA) and contains approximately 62 % by weight or 38 % by volume inorganic fillers such as Ba-Al-F silicate glass and SiO₂. The filler particle size is between 0.005 µm and 5 µm.

Applications:

- enlarged fissural sealing
- fillings of class V according to Black
- minimal invasive fillings of classes I and II according to Black in the area which is not strongly stressed by chewing
- minimal invasive fillings of class III according to Black
- slighter corrections of form and shade at the enamel
- splinting of teeth which are traumatically loose
- adjustments of developmental teeth defects like e.g. enamel hypoplasia
- flexible sealer layer, cavity lining for Black-classes I and II
- bracket adhesive

Contraindications:

This product or one of its components may in particular cases cause hypersensitive reactions. If suspected, information on the ingredients can be obtained from the manufacturer.

Kind of application:

In order to prepare the treatment it is recommended to clean the hard dental substance by means of a non-fluoride polishing paste. Prior to starting the treatment the shades should be selected from the attached colour scale as long as the teeth are still humid.

The application of cavity lining materials (e.g. glass-ionomer-cement, calcium hydrate preparations) is normally not necessary.

The enclosed disposable metal cannulas are suitable for the application.

1. Preparation of cavity according to the general regulations of the composite filling therapy with gentle treatment of the hard dental substance. Then clean and dry the cavity. It is recommended to use rubber dam.
2. Etching of enamel by means of GLUMA® Etch 20 or GLUMA® Etch 35. If dentin had been exposed during the preparation it becomes necessary to apply an adequate adhesive system (e.g. GLUMA® Comfort® Bond, GLUMA® Solid Bond of Kulzer) prior to applying the filling material to the cavity. Avoid contamination of saliva!
3. Apply FLOW LINE to the cavity in thin layers (max. 2 mm). When applying by means of a cannula do not insert air bubbles. Take care of a sufficient moistening of the prepared hard dental substance.
4. Polymerize each layer according to the table for a period of 20 or 40 s using a Translux® light device. During polymerization a dispersive layer is built up at the surface which may not be touched or removed. It serves as connection to the following composite layer.

Shade	Time
A1, A2, A3, A3.5, B2, B3	20 s
OA2, OA4, OB2	40 s

5. After polymerization FLOW LINE may immediately be processed and polished. For processing finishing diamonds, flexible discs, silicone polishing devices as well as polishing brushes are suitable.

During restorations requiring much time the operating light should temporarily be removed from the working area in order to prevent a premature curing of the composite.

Precautions:

The periods mentioned in the instructions for use are minimum periods which should not be reduced in order to achieve the optimal filling quality.

The disposable cannulas attached to the syringes are meant which have to be disposed after each application. After the material has been yielded from the syringe relieve the plunger. Replace the cap of syringe after each use.

In case of direct contact with dentin which is adjacent to the pulp irritations may not be excluded.

If necessary protect the dentin adjacent to the pulp by an appropriate cavity lining. Apply the smallest possible amount of a calcium hydrate preparation to the parts adjacent to the pulp only ($\varnothing < 0,5$ mm) and cover with glass-ionomeric cement. Do not etch glass-ionomeric cement!

In order to avoid the etching of glass-ionomeric cement it is recommended to perform the etching prior to the application of the calcium hydrate preparation and of the glass-ionomeric cement.

Do not use in connection with preparations containing eugenol because these may impede the curing.

The material irritates eyes as well as skin. Thus sensitizations may be caused by skin contact.

Therefore, avoid contact with skin and eyes. In case of contact with eyes immediately rinse with plenty of water and seek medical advice if necessary.

Do not use the material after the expiry date. Do not store above 25°C (77°F). Keep out of direct sunlight.

Delivery units:

Packing units:

Single packages as syringes or PLTs

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Kulzer GmbH
Kulzer, LLC