CHARISMA®

9606182/111

CHARISMA®

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GB Instructions for use

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SE Bruksanvisning

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NO Bruksanvisning

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CA Instructions for use

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CHARISMA®

Product Description
Charisma is a light curing, radio-opaque microhybrid composite that is using Microglass® technology. It is indicated for adhesively bonded, tooth coloured restorations. It is suitable for anterior and posterior applications.

Composition
Charisma is based on a BIS-GMA matrix and contains approximately 61% filler by volume, with 60% inorganic filler by volume and a particle size of 0.005–10 µm.

- Barium Aluminium Fluoride glass
- Feldspar
- Pre-polymerized filler

Indications

- Direct restoration of Class I–V cavities (according to G.V. Black)
- Direct composite veneers
- Shape corrections of teeth (i.e. diastemas, congenital defects in teeth, etc.)
- Splitting of teeth loosened by trauma or periodontal disease
- Restoration of primary teeth
- Repairs of porcelain, composite (in combination with an adequate repair-system)

Contraindications
Application to dentine close to the pulp (less than 1 mm). Do not use in case of a known allergy to any of the constituents of Charisma. The use of this product is contraindicated in case of known or suspected allergies against (meth)acrylate compounds.

Side effects
This product or one of its components may in particular cases cause hypersensitive reactions. If suspected, information on the ingredients can be obtained from the manufacturer.

Precautions:

- Avoid contact with skin. Wear suitable protective gloves. In case of contact with skin, immediately wash with plenty of water and soap. May cause sensitisation by skin contact. Avoid contact with eyes. Immediately and thoroughly rinse with plenty of water in case of contact with eyes and contact an ophthalmologist.
- In the case of deep cavity preparations in close proximity to the pulp or a direct pulpal exposure, cover the dentine or the exposure locally with a calcium hydroxide liner covered with a glass ionomer cement before applying the adhesive.
- Do not use eugenol containing materials in or around the cavity preparation being restored, because these may interfere with the proper curing of Charisma.
- The PLTs (pre-loaded tips) are intended for single use only and cannot be sterilized or disinfected without damaging and contaminating the restorative material because the solutions contain phenolic constituents. To avoid cross infection, items intended for single use must be disposed of immediately after initial use.

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- When using Charisma PLTs, use a gentle, even motion when exerting pressure on the applicator gun. Excessive force or sudden movement could create a potential hazard upon extrusion.
- Use adequate protection for syringes to prevent contamination. Do not reuse after contamination.
- Visually inspect for damage before first use. Damaged products must not be used. Composites should only be used at room temperature.
- Composites should only be used when having reached room temperature.

- Application**
Select an appropriate shade or shade combination from the Charisma Shade Guide prior to placing the rubber dam. This is important because teeth will significantly change colour when desiccated. If the tooth surface being treated is not to be instrumented by a diamond stone, carbide bur, drill or other abrasive instrument, the surface should be cleaned with pumice or a fluoride free polishing paste before restoration is placed. A non-sticking plastic, Teflon coated or gold plated cleaned instrument (such as the Kulzer Plasmacat Instrument) is recommended for use in clinically placing, sculpting and shaping the material.
- For a similar result, use Ivory® rubber dam similar to control contamination from moisture, blood and saliva.
 - Prepare the cavity according to generally accepted rules for the clinical composite application. Then clean and dry the cavity.
 - Prior to application of Charisma restorative material for the cavity preparation, pre-treatment of the tooth structure with an adequate enamel/dentine adhesive is mandatory (e.g. IBOND® Self Etch, IBOND® Total Etch, GLUMA® Solid Bond by Kulzer). Make sure that no excess remains in the corners of the cavity. Once treated with the adhesive, avoid contamination with saliva or blood. The use of a flowable composite material (e.g. Venus® Diamond Flow) allows the creation of a smooth cavity floor and facilitates the adaptation of the filling material. When using a layer of a flowable material, apply it according to the specifications in the respective instructions for use and cure it separately.
 - Apply Charisma restorative material in thin layers (max. 2 mm) and adapt carefully to the tooth surface or cavity walls. In case of severe discoloration of the underlying tooth or surface being restored, it may be necessary to mask this by using a thin layer of one of the recommended Charisma opaque dentine shades first.
 - To avoid excess translucency and achieve an ideal aesthetic result, the case of large Class IV cavities and anterior tooth reconstruction, it is necessary to utilise a Charisma opaque dentine shade (max. 1 mm) in the deepest layers, which can then be covered by the universal Charisma shades if necessary.
 - Polymersise Charisma in thin layers (max. 2 mm) according to the table below using an acceptable QHT (Quarz-Halogen-Tungsten) or LED curing light such as the Kulzer Translux® light curing unit or any other blue-light unit (wavelength maximum at 460–470 nm; light output of 1550–600 mW/cm²). During polymerisation, an inhibited-layer forms on the surface which should not be touched, contaminated or removed during the build-up phase of the restoration. It serves as chemical connection for any subsequent composite layers that are added.

Charisma Shades: recommended curing time and maximum layer thickness for Translux®-curing units or other blue-light-curing units.

Universat: A1, A2, A3, A3.5, A4, B1, B2, B3, C2, C3, C4, D3	20 s	2 mm
Opaque: OA2, OA3, OA3.5	40 s	1 mm

The data given is correct to the best of our knowledge and are based on internal test results.

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