# GLUMA® Desensitizer PowerGel

DE	Gebrauchsanweisung.....	3
GB	Instructions for use.....	7
FR	Mode d'emploi .....	11
ES	Instrucciones de uso .....	15
IT	Istruzioni per l'uso .....	19
PT	Instruções de uso .....	23
NL	Gebruiksaanwijzing.....	27
SE	Bruksanvisning .....	31
DK	Brugervejledning .....	35
NO	Bruksanvisning .....	39
FI	Käyttöohjeet .....	43
GR	Οδηγίες Χρήσης.....	47
CZ	Návod k použití .....	51
HU	Használati utasítás .....	55
LV	Lietošanas instrukcija.....	59
LT	Naudojimo instrukcija.....	63
PL	Instrukcja obsługi .....	67

**KULZER**  
MITSUI CHEMICALS GROUP

# GLUMA® Desensitizer PowerGel

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DE

## Produktbeschreibung

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GB

## Product description

GLUMA Desensitizer PowerGel is used for the treatment of hypersensitive dentine. It achieves its effects by precipitation of plasma proteins, which reduces dentinal permeability and occludes the peripheral dentinal tubules. This inhibits the flow of fluid through the tubules which is the cause of sensitivity.

## Composition

- (2-hydroxyethyl)methacrylate • glutardialdehyde • pyrogenic silicic acid • purified water

## Indications

- To reduce or even eliminate pain in exposed cervical areas not requiring restoration.
- To alleviate or prevent dentinal sensitivity after preparation of teeth to receive fixed prosthesis or restorations.

## Contra-indications

GLUMA Desensitizer PowerGel must not be used:

- if the necessary precautions cannot be taken or the specified application technique cannot be used
- the use of this product is contraindicated in case of known or suspected allergies against GLUMA Desensitizer PowerGel; in case of suspected allergies, an allergy test is recommended prior to treatment
- if the pain is due to pulpitis
- if the pulp is exposed or the dentine adjacent to the pulp is perforated

## **Side effects**

This product or one of its components may in particular cases cause hypersensitive reactions. If suspected, information on the ingredients can be obtained from the manufacturer.

## **Warnings / Precautions**

Irritating to respiratory system. Harmful by inhalation. GLUMA Desensitizer PowerGel contains glutaraldehyde which can cause local chemical burns or may cause sensitisation through direct contact or inhalation. The product contains (meth)acrylate compounds that may cause sensitisation in case of skin contact.

Wear suitable protective gloves. Skin irritant. Avoid contact with skin. In case of contact with skin, immediately wash with plenty of water and soap. Risk of serious eye injury in case of eye contact. Irritating to eyes. Avoid contact with eyes. Immediately and thoroughly rinse with plenty of water in case of contact with eyes and contact an ophthalmologist. Do not swallow or ingest this product. Immediately contact a physician if health-related complaints arise after swallowing or aspirating the product. Wear safety glasses and ensure patient has adequate face protection.

To avoid cross infection, items (enclosed cannulae) intended for single use must be disposed of immediately after initial use.

## **Dosage and Administration**

### **Treatment of hypersensitive dentine**

The treatment of hypersensitive dentinal surfaces is only indicated in cases of severe pain in reactions to thermal, osmotic or tactile stimuli. In cases of mild pain, alternative treatments such as desensitizing toothpastes should be used.

- Clean the dentine (e.g. with cleansing pumice), under local anesthesia if necessary, and then rinse off with water.
- Protect mucous membranes from contact with the product by using a rubber dam.

- Apply the **smallest possible amount** of GLUMA Desensitizer PowerGel required for one treatment to the dentinal surface and then leave for 30–60 seconds.
- Then rinse thoroughly with water and apply suction.
- In the case of deep cavity preparations in close proximity to the pulp or a direct pulpal exposure, cover the dentine or the exposure locally with a calcium hydroxide liner covered with a glass ionomer cement before applying the GLUMA Desensitizer PowerGel.

If a single application of GLUMA Desensitizer PowerGel does not produce sufficient and sustained pain relief, the treatment can be repeated at the same or a subsequent appointment before alternative treatments are used.

#### **Treatment of dentine prepared to receive fixed dentures or complete cast restorations**

- Protect mucous membranes from contact with the product by using a rubber dam.
- Remove moisture from the prepared dentine by applying a gentle stream of compressed air.
- Apply GLUMA Desensitizer PowerGel and leave for 30–60 seconds. Make sure that only the **smallest possible amount** required is applied.
- Then rinse thoroughly with water and apply suction.
- In the case of deep cavity preparations in close proximity to the pulp or a direct pulpal exposure, cover the dentine or the exposure locally with a calcium hydroxide liner covered with a glass ionomer cement before applying the GLUMA Desensitizer PowerGel.

#### **Important instructions for use**

Visually inspect for damage before first use. Damaged products must not be used. Remove only the smallest quantity necessary and close the container immediately. GLUMA Desensitizer PowerGel must not come into contact with metal (e.g. cannulae or other application aids).

**Special instructions**

Only to be used by dentists and for its intended use.

**Storage**

Keep out of direct sunlight. Do not store above 25°C (77°F).

Do not use the material after the expiration date.

Please quote batch number and expiry date in all correspondence about the product.

Expiration date  $\times$  and batch number  $\boxed{\text{Lot}}$ : See note on syringe/packaging.

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Kulzer GmbH
Kulzer, LLC