Preciano® Silberleitlack TK

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Made in Germany

KULZER
MITSUBI CHEMICALS GROUP

Preciano® conductive silver varnish TK

Hera Preciano conductive silver varnish TK is a thin consistency, rapid drying lacquer containing silver. It is used for producing electroconductive surfaces for electroforming restorations. It does not react with Hera Preciano Bio Gold solution universal.

Application: Shake conductive silver varnish TK carefully before use until the mixing balls can be heard. Place a small amount in the dropper lid and use a Hera Preciano special brush to apply a thin uniform, masking coat of varnish without allowing it to puddle. The metal surface being electroformed must have been degreased, cleaned and dried in advance – the contact rod, which is adhered into place, must also be coated. Only use minute amounts of varnish to touch up defective areas in the coating. Allow to dry for at least 30 minutes at room temperature. After removing conductive silver varnish TK from the bottle, cover it immediately with foil (Adapta) etc. to prevent the varnish drying out. Once finished, close the bottle tightly and clean the Hera Preciano special brush with brush cleaner.

Once electroforming has been completed, the conductive silver varnish TK can be dissolved in an ultrasonic cleaner (max. temperature: 30 °C (86 °F)) using Hera Preciano conductive silver varnish remover.

Please note: As thick conductive silver varnish may lead to uneven coatings, discontinue using it. Please do not thin!

Further information: To deodorise during drying airing is required.

For Information on the use of conductive silver lacquer for telescopic crowns and bar-borne restorations, please refer to currently valid Hera Preciano instructions for use.

Storage: Seal the container airtight immediately after use, place it in the packaging and keep it in a well ventilated place, protected from sunlight.

Shelf-life: See imprint on paper box and bottle.

Dated: 2017-07

Hazard-determining components: 1-methoxypropan-2-ol

Hazard statements

Flammable liquid and vapour. May cause drowsiness or dizziness.

Precautionary statements

Keep away from heat / sparks / open flames / hot surfaces. – No smoking. Use explosion-proof electrical / ventilating / lighting / equipment. Avoid breathing dust / fume / gas / mist / vapours / spray. Wear protective gloves / protective clothing / eye protection / face protection. **IF ON SKIN (or hair):** Remove / Take off immediately all contaminated clothing. Rinse skin with water / shower. Store locked up.

Warning

Manufacturer:
Kulzer GmbH
Leipziger Straße 2
63450 Hanau (Germany)
Made in Germany

KULZER
MITSUBI CHEMICALS GROUP