Aislar[®]

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GB Instructions for use

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SE Bruksanvisning

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GB Instructions for use Aislar[®]

Composition Insulation, alginate-based, free of formaldehyde	Storage advice Do not store above 25°C (77°F). Do not use after expiry date. Avoid direct exposure to sun. When product has been removed, reseal carefully. Keep away from children. Visually inspect for damage before first use. Damaged products must not be used.
Indications Suitable for isolating plaster models using the acrylic technique. The isolation film ensures smooth surfaces on the acrylic. Perfect separation of acrylic/plaster and plaster/plaster is assured.	Disposal information Observe the material safety data sheet or national regulations (Available at www.kulzer-dental.com or from your Kulzer contact).
Contraindications Not for use in the mouth.	® = registered trademark of Kulzer GmbH
Advice Avoid contamination of the isolation liquid, e.g. due to plaster residues, metal ferrule brush etc.	Dated: 2015-06
Handling The wax must be thoroughly scalded. Place the required quantity of Aislar into a jar and apply with a brush. Coat the warm or cold plaster twice with a thin insulation film after scalding. No residue should remain in grooves and furrows. Acrylic teeth and clamps should not be wetted. After drying the isolation film, the acrylic dough can be inserted immediately. Surplus Aislar already poured out must not be returned to the original bottle. With dried out models all plaster parts must be sufficiently watered prior to insulation. The watering time is subject to the degree of dryness and is approximately 5–15 minutes. Watering prevents evaporation of air from plaster into the resin during polymerisation under pressure.	

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© = indregistreret varemærke fra Kulzer GmbH