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GB Instructions for use

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Manufacturer:
Kulzer GmbH
Leipziger Straße 2
63450 Hansau (Germany)

Made in Romania

Distributed in USA / Canada exclusively by:
Kulzer, LLC
4315 South Lafayette Blvd.
South Bend, IN 46614-2517
1-800-431-1785

Caution: Federal law restricts this device to sale by or on the order of a dental professional.

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Lab Putty 65/90

Instructions for use **GB**

Pala Lab Putty 65 and Pala Lab Putty 90 are kneadable polyvinyl siloxane materials, for use in dental laboratories. Mixing ratio 1:1, addition curing.

Fields of Application

- Pala Lab Putty 65/90 can be used for
- Matrix for construction of provisional
 - Matrix for setting anterior teeth in an immediate denture
 - Matrix for construction of bars
 - Matrix for processing teeth on a partial denture
 - Duplicating models
 - Creating preparation models for the repair
 - Insulation of teeth against plaster
 - Matrix for fabrication of hybrid over dentures

Contra-indications

Do not use Pala Lab Putty 65/90 on patients.

Side effects

Allergies to the product or its constituents may occur in isolated cases. If an allergy is suspected, request a list of constituents from the manufacturer.

Notes on use

Close tubs properly after use. Do not mix up caps/lids/spoons. Cross-contamination of the components will cause partial curing and make the material unusable.

We recommend the use of vinyl gloves or nitrile gloves for single use while processing the material. If latex gloves are used they must be checked for compatibility prior to use as like true for all addition-curing silicone impression materials, the setting process may be affected by contact with products containing active Sulfur or Nitrogen.

Pala Lab Putty 65/90 materials should be at room temperature (23°C (73°F)) for processing. Higher temperatures shorten, lower temperatures prolong the times stated below. Extensive hand mixing may accelerate setting time due to body and frictional heat. Polyvinyl siloxanes are chemically resistant when cured and cannot be removed from clothes after soiling. Material which has not set can soil clothing.

Working and setting times

	Mixing time s	Working time* min:s	Setting time* min:s	Hardness Shore A
Pala Lab Putty 65	30	1:30	6:00	60-70
Pala Lab Putty 90	45	1:30	6:00	85-95

* related to room temperature of 23°C (73°F)

Application

- Dose equal amounts (1:1) using the enclosed spoons (colored spoon for colored component, white spoon for white component). Over dosage of catalyst (white) will not cause faster setting.
- Hand mix both components until a homogenous color is achieved.

Warnings and safety instructions

If material comes into contact with the eyes, rinse it out with water for several minutes. If symptoms persist, consult an eye specialist.

Storage note

Do not use Pala Lab Putty 65/90 materials if the expiry date (see label) has elapsed. Do not store above 25°C (77°F). Keep out of children's reach.

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