Palabond®

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GB Instructions for use

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SE Bruksanvisning

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GB Instructions for use Palabond®

Product Description

Adhesive primer for bond enhancement between denture base resins and highly crosslinked acrylic teeth.

Composition

Methylmethacrylate, Dimethacrylate, Methacryl acid

Processing

To improve the bond of the base resins and the acrylic tooth, after isolating the plaster model with Aislar®, the basal surfaces of the tooth are roughened with a coarse diamond burr or by sand-blasting and the grinding dust is removed. Apply the Palabond with a brush (no metal ferrule) twice and allow it to work in for 30 seconds each time. After the second coat the adhesion primer remains active for 10 minutes. Then process the prosthetic acrylic in accordance with the respective information for use.

Repair

For repair of used dentures roughen surfaces (course diamond-burr) to be connected and treat with Palabond as mentioned above.

Caution:

After application, Palabond remains active for 10 minutes. Finishing or repairing the denture must be carried out within this time.

Storage advice

Do not store above 25°C (77°F). Do not use after expiry date. Avoid direct exposure to sun. When product has been removed, reseal carefully. Keep away from children. Visually inspect for damage before first use. Damaged products must not be used.

Hazard/Safety information

Highly flammable liquid and vapour. Causes skin irritation. Causes serious eye damage. May cause an allergic skin reaction. May cause respiratory irritation. Keep away from heat/sparks/open flames/hot surfaces. – No smoking. Use explosion-proof electrical/ventilating/lighting/equipment. IF ON SKIN (or hair): Remove/Take off immediately all contaminated clothing. Rinse skin with water/shower. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Immediately call a POISON CENTER/doctor. Store locked up.

Safety data sheets and more information are available at our website www.kulzer.com

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® = registered trademark of Kulzer GmbH
Kulzer GmbH