Palaferm®-Isolierung

66033287/11

- DE** Gebrauchsanweisung
- GB** Instructions for use
- FR** Mode d'emploi
- ES** Instrucciones de uso
- IT** Istruzioni per l'uso
- PT** Instruções de uso
- NL** Gebruiksaanwijzing
- SE** Bruksanvisning

Manufacturer:
Kulzer GmbH
Leipziger Straße 2
63450 Hanau (Germany)

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Status: 2016-01

GB Instructions for use Palaferm®-Isolierung

For the isolation of plaster and resin models to methacrylate resins.

Color: Clear

Handling

Fully clean the model, removing wax and other residues. Place the required quantity of material into a vessel. If isolating resin-based models apply a single coat of Palaferm with a soft brush. For gypsum models apply two coats of Palaferm with a soft brush. Wait for the first layer of isolation to dry completely before application of the second coat. An optimal isolation can be achieved by even application onto a cooled down plaster / resin-based model. Palaferm dries within 2–4 minutes depending on the thickness applied and ambient temperature. After complete drying, denture acrylic should be applied immediately. Be careful to avoid pooling and creating excess residue.

Advice

Dried out plaster models must be hydrated with water prior to isolation (depending on the dryness approx. 5–15 min). Wetting prevents air bubbles from penetrating the resin during the pressure polymerization process.

Storage

Do not use material after expiry date. Do not store above 25°C (77°F). Avoid direct exposure to sunlight. Always store in its original container. After decanting, do not place waste material back into the container.

Warning advice

Contains highly inflammable solvents.

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