# Palajet® Duoflask / singleflask

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## GB Instructions for use

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## SE Bruksanvisning

## FI Bruksanvisning

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Abb. 1 / fig. 1

Abb. 2 / fig. 2

|   |   |   |   |             |
|---|---|---|---|-------------|
|   | <b>KULZER</b><br>MITSUI CHEMICALS GROUP |  | <p>Manufacturer:<br/>Kulzer GmbH<br/>Leipziger Straße 2<br/>63450 Hanau (Germany)</p> <p>Distributed in USA /<br/>Canada exclusively by:<br/>Kulzer, LLC<br/>4315 South Lafayette Blvd.<br/>South Bend, IN 46614-2517<br/>1-800-431-1785</p> <p>Caution: Federal law restricts this device to sale by or on the order of a dental professional.</p> |             |
| <p>Importado e Distribuído por<br/>Kulzer South América Ltda.<br/>CNPJ 48.708.010/0001-02<br/>Rua Cenzo Sbrighi, 27 – cj. 42<br/>São Paulo – SP – CEP 05036-010<br/>sac@kulzer-dental.com<br/>Resp. Técnica: Dra. Regiane Marton –<br/>CRO 70.705<br/>Nº ANVISA: vide embalagem</p> |   |   |   | 99000672/11 |

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## GB Instructions for use Palajet® Duoflask / singleflask

**Application field:**

Flask for simultaneous production of one or two full dentures. Palajet Duoflask / singleflask is a component of the Palajet compressed air injection system.

**Procedural steps**

**Preparation**

The dentures should to be set up in wax and waxed up anatomically to reduce finishing time.

**Investing**

Prior to investing, make sure both flask-halves have the same matching number to ensure a precise fit. Apply Vaseline to the inner sides so that the plaster can be removed more easily later on.

The dentures that are set up in wax must be invested with the arch facing to the outside (figure 1).

Remove excess stone and produce a smooth surface. Any stone residue must be removed from the edges of the flask. Use stone of class III for investing. After hardening of the stone, use wax to attach the injection canal (diameter 7 mm PalaXpress® canal wax) and the vent canal (diameter 3 mm PalaXpress® canal wax). The injection canal must be attached to the tuberosity of the maxillary or the retromolar pad of the mandibular. The vent exit canal is attached to the opposite tuberosity or retromolar pad (fig. 1).

Apply Aislar® separating agent to the surface of the stone while flask is warm. Place the flask halves onto each other and tighten the four eye bolts crosswise. Pour stone through the two openings and fill up until the stone reaches the occlusal surfaces of the teeth and let the stone harden. Apply Aislar® separating agent and fill remainder of flask up with stone. Remove excess material (stone) so that the flask can be fitted between the centring discs.

**Processing and injection of the resin:**
Please find the processing instructions in the current instructions for use of the relevant acrylic. Prior to screwing the filled filling cylinder into the Palajet®, the spacer must be placed on the filling cylinder (see fig. 2). The spacer is used for single and double cувettes, but NOT for the old brass cувettes!

The remaining steps of the procedure can be taken from the Palajet instructions for use.

**Information on cleaning and care:**

The surface of the flask features an extremely resistant anodised aluminium coating. Do not damage the layer with any sharp object (e. g. knife or hammer). Remove plaster residue with water or steam. Depending on the degree of usage, the seat of the eye bolts must be cleaned within regular intervals and greased with Vaseline. To allow easier cleaning, the eye bolts can be removed by unscrewing the hinge screws.

**Exclusion of liability:**
The manufacturer does not accept any liability for use that is not described in the instructions for use as well as for arbitrary modifications resp. damage of the flask.

**Delivery items:**
Flask with spacer and wrench.

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© = registrerat varumärke för Kulzer GmbH
® = registered Trademark of Kulzer GmbH
© = rekisteröity tavaramerkki, jonka omistaa Kulzer GmbH