## Palatray® XL

- (DE)** Gebrauchsanweisung
- (GB)** Instructions for use
- (FR)** Mode d'emploi
- (ES)** Instrucciones de uso
- (IT)** Istruzioni per l'uso
- (PT)** Instruções de uso
- (NL)** Gebruiksaanwijzing
- (SE)** Bruksanvisning
- (DK)** Brugervejledning
- (NO)** Bruksanvisning
- (FI)** Käyttöohjeet
- (GR)** Οδηγίες χρήσης
- (PL)** Instrukcja obsługi

## (GB) Instructions for use

### Palatray® XL

#### Product description

Palatray XL is a rigid, one component light curing composite with high dimensional stability supplied in anatomically shaped and enlarged sheet form.

#### Composition

High molecular, multifunctional meth-/acrylates and inorganicfiller.

#### Indications

- individual trays
- functional trays
- base plates for bite wings
- base plates for wax-ups

#### Contraindications

If the patient is allergic to one or more ingredients of Palatray XL, the product must not be used.

#### Advice

For use as directed in the dental laboratory only. The product contains (meth)acrylate compounds that may cause sensitisation by skin contact. Discontinue using the product if symptoms of irritation occur. Do not use the product in case of known allergies to resins based on acrylates or methacrylates.

#### Handling

Mark the desired margin line on the model and block-out under-cut areas. Water the model,apply a thin layer of Aislar® and allow to dry. Adapt a sheet of Palatray XL on to the model exactly and reduce the edges to the margin line. Use excess material to form a handle. After polymerizing, remove the tray from the model and trim with carbide burrs. Use a dust extractor for trimming.

#### Polymerizing

HiLite® power:	180 s
Heraflash®:	180 s
UniXS®:	180 s

Large models that do not fit into the curing container (e.g. UniXS) can be placed directly into the curing chamber.

#### Hazard/Safety information

May cause an allergic skin reaction. Avoid breathing dust/fume/gas/mist/vapours/spray. Wear protective gloves/protective clothing/eye protection/face protection. Wash contaminated clothing before reuse. If skin irritation or rash occurs: Get medical advice/attention. IF ON SKIN: Wash with plenty of water.

#### Storage advice

Do not store above 25°C (77°F). Do not use after expiry date. Avoid direct exposure to sun. When product has been removed, reseal carefully. Store out of reach of children. Visually inspect for damage before first use. Damaged products must not be used.

Safety data sheets and more information are available at our website [www.kulzer-dental.com](http://www.kulzer-dental.com)

\* = registered trademark of Kulzer GmbH

Dated: 2016-01

Distributed in USA /

Canada exclusively by:

Kulzer, LLC

4315 South Lafayette Blvd.

South Bend, IN 46614-2517

1-800-431-1785

Caution: Federal law restricts this device to sale

by or on the order of a dental professional.

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\* = marca comercial registrada da Kulzer GmbH