<b>DE</b>	Gebruuchsanweisung
<b>GB</b>	Instructions for use
<b>FR</b>	Mode d'emploi
<b>ES</b>	Instrucciones de uso
<b>IT</b>	Istruzioni per l'uso
<b>PT</b>	Instruções de uso
<b>NL</b>	Gebruiksaanwijzing
<b>SE</b>	Bruksanvisning
<b>DK</b>	Brugervejledning
<b>NO</b>	Bruksanvisning
<b>FI</b>	Käyttöohjeet
<b>GR</b>	Οδηγίες χρήσης
<b>CZ</b>	Návod k použití
<b>HU</b>	Használati utasítás
<b>LV</b>	Lietošanas instrukcija
<b>LT</b>	Naudojimo instrukcija
<b>PL</b>	Instrukcja obsługi
<b>TR</b>	Kullanma talimatı
<b>RU</b>	Инструкция по применению
<b>UA</b>	Інструкція по застосуванню

## KULZER

MITSUBI CHEMICALS GROUP

Manufacturer:
 Kulzer GmbH
 Leipziger Straße 2
 63450 Hanau (Germany)

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## GB Instructions for use Palavit® G

**Composition**

Methyl methacrylate-based cold-curing polymer in the form of powder and liquid. The catalyst system contains tertiary amine. Palavit G is cadmium-free.

**Areas of application**

For modelling in cast crowns and bridges.

**Advice**

Not for use in the mouth.

**For calculating alloy weight**
Density of Palavit G = 1.2 g/cm<sup>3</sup>.

**Properties**

Short, reliable curing process, high precision, rigid, fusion temperature approx. 110°C (230°F), burns without residue.

**Application**

**Brush method**

A brush saturated with liquid is immersed in prepared powder. The dissolved Palavit G is then ready for use.

**Castable dough**

Introduce Palavit G liquid to the stirring vessel and sprinkle in powder until a low level of liquid remains. Powder-to-liquid mixing ratio approx. 5:3.

**Mouldable dough**

Stir Palavit G in the mixing vessel to make a thick dough. After the transition to the mouldable plastic state the dough can be shaped and applied.

**Polymerisation**

Curing time approx. 10 minutes; curing time depends on the method of working and ambient temperature.

**Storage advice**

Do not store above 25°C (77°F). Do not use after expiry date. Avoid direct exposure to sun. When product has been removed, reseal carefully. Store out of reach of children.

**Hazard information**

Highly flammable. Harmful to health if inhaled, swallowed or allowed to come into contact with the skin. Danger of cumulative effects. Irritates the eyes. Irritates the respiratory system. Irritates the skin. (Irritates the mucosa). Sensitivity by skin contact is possible. Keep container in a well-ventilated place. Keep away from sources of ignition – no smoking. Take precautionary measures against static discharges. Wear protective clothing. In case of fire, use sand, carbon dioxide or powdered extinguishing agent. Never use water. Do not inhale vapours.

**Disposal information**

Regarding disposal, please observe national regulations and the safety data sheet. (Available at [www.kulzer-dental.com](http://www.kulzer-dental.com) or from your Kulzer contact.)

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® = registered trademark of Kulzer GmbH