PreVISION® Temp

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Temporary Crown & Bridge Material

Product Description

PreVision Temp is an auto mixed two-component, self curing provisional dental restorative material based on multifunctional methacrylic esters.

Indication

Fabrication of temporary crowns, bridges, inlays, onlays and veneers. Intended for use as a short and long term temporary material.

Contra Indication

Do not use for patients with allergic reactions to acrylates.

Side effects

PreVision Temp contains methacrylates. Rarely, with susceptible patients, sensitization to PreVision Temp cannot be excluded. In these cases, or if allergic reactions are observed PreVision Temp should not be used.

Precautions

Avoid contact with skin, mucous membrane and eyes. On contact with the eyes, rinse the eyes immediately with copious amounts of water and seek medical advice if required. On contact with the skin, wash off immediately using water and soap. After swallowing consult a doctor immediately.

Impression taking

Take a eugenol free impression before preparing teeth or before a planned tooth extraction. Use a silicone impression material, like Flexitime® or Xantasil® or an alginate. Carve out interdental areas where needed. In molar areas with absent teeth cut a groove in the impression between the abutments to create a bridge-like connection to stabilize the abutments.

Preparing the cartridge

Remove the safety cartridge turn cap and discard. Attach Mixing Tip blue (10:1) with a sideways turning movement, locking the mixing tip in place. Load the application gun with the prepared cartridge to begin application. Before a mixing tip is applied for the first time, squeeze paste out onto a mixing block until the base and catalyst pastes are the same. Discard the paste which has been squeezed out.

Nota: For each new mix, discard approximately 1–2 mm (size of a pea) of product from the mixing tip. Leave the used mixing tip on the cartridge to serve as a cap until the next use. To avoid cross infection, tents intended for single use must be disposed of after use.

Application

PreVision Temp is automatically mixed when dispensed with slight and even pressure directly into the preliminary impression made before the preparations. Filling should occur from bottom upward to prevent voids.

Forming the temporary crowns or bridges

– Load the preliminary impression with PreVision Temp.

– Seat the impression onto the prepared areas of the teeth. Steps a and b must be done within 50 sec (working time).

– After 1–2 min (setting time in mouth) the material shows a hardened but still elastic condition and can be removed from the teeth (alternatively: 3–4 min setting time on the bench at room temperature 23°C (73°F)).

Nota: The material can only be removed, without distortion, during the elastic state.

Time Table for application of PreVision Temp

– 00:00–00:50 – Dispense material into preliminary impression and insert into mouth.

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