# Venus® Supra

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## Venus® Supra

**Composition**  
The polishers are latex-free and made of synthetic rubber with impregnated microfine diamond powder and color pigments (prepolisher).

**Indication**  
The diamond polishers are ideal for polishing finished composite restorations.

**Use**  
The restoration intended for polishing must be properly finished, as well as being smooth and free from defects. The instruments are to be brought up to their running speed before being used on the restoration. The maximum rotational speed is 10,000 rpm. You will obtain the best results with a rotational speed of around 7,500 rpm. A water spray is necessary to dissipate the heat (minimum flow of water 50 ml/min). To avoid higher temperatures you should work with a low contact pressure. The maximum permitted rotational speed should never be exceeded. The polishers tend to vibrate when the maximum permitted rotational speed is exceeded which can lead to damage to the polisher, bending of the shaft and/or breakage of the instrument, i.e. constituting a danger to the user, the patient or third parties. Maintaining the correct rotational speed range leads to the best results. Deformed instruments and instruments that do not run true must be immediately disposed. Always wear eye protection. A respirator must be worn in order to prevent the inhalation of airborne particles. Kulzer diamond polishers meet ISO standards and can be used in commercially available hand pieces and contra-angles provided they are in a proper condition for use. The diamond polishers must be placed in the hand pieces or contra-angles carefully and without using force. They must be axially mounted as deeply as possible.

**Warning Notices**  
Failure to heed the maximum permissible rotational speed leads to an increased safety risk. With insufficient cooling water, irreversible damage can be caused to the tooth and surrounding tissue. After use in the mouth the polishers must be immediately disinfected and sterilised. Afterwards cleaning and repeated disinfection and sterilisation must be carried out. In reprocessing contaminated instruments it is essential to wear protective clothing and gloves.

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**Preparation and Reprocessing Instructions**  
All instruments are delivered unsterile and must be run through the indicated cycle before and after each use. The label on the cleaning and/or disinfecting solution must specifically say "suitable for rubber polishers or synthetics/silicones", because if the label only says "rotary instruments", the solution may not be suitable for polishers. Strong acids and strong bases may oxidize the stainless steel shaft. Rinse the polisher with distilled water after the treatment with cleaning and disinfectant solutions. Avoid temperatures >150° C (302°F). Ultrasonics bath must not exceed temperatures >150° C (302°F) because of the possible coagulation of albumen.

**I. Cleaning Preparation:** Pre-clean under running water with a brush (plastic) directly after use.

**II. Cleaning Manual:** Under running water with a brush (plastic bristles).

**Cleaning Machine assisted:** 1. Ultrasonic assisted cleaning with a suitable cleaning agent and disinfectant. 2. Rinse the instruments afterwards under running water.

or

**Cleaning Automatic:** Thermal disinfection: Manufacturer's specification according to DIN EN ISO 15883. Cleaning programme as indicated by the manufacturer in the operating instructions.

**III. Disinfection:** Use a solution classified as suitable for rubber and silicone polishers and according to the disinfectant manufacturer. Exposure times and concentrations recommended by the manufacturer should be adhered to.

**IV. Rinsing:** Rinse the polishers with distilled water after treatment with cleaning and disinfectant solutions.

**V. Maintenance:** Visual check of all instruments with optical magnification (5–10 fold)

**VII. Inspection:** 1. No residues = continue to sterilization. 2. Visible residues = repeat cleaning. Reject and dispose of instruments in the event of discernible defects.

**VIII. Sterilization:** Steam sterilizer: Appliance according to EN 13060, validated procedure.

Category S- or B Sterilizer.
1. Holding time: Full cycle 5 min

2. Sterilisation temperature 134° C (273°F)

3. Drying time 10 min

4. Threshold values of contents for feed-water and steam condensates

5. Loading of sterilizer according to manufacturer's instructions

6. Follow manufacturer's operating instructions

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